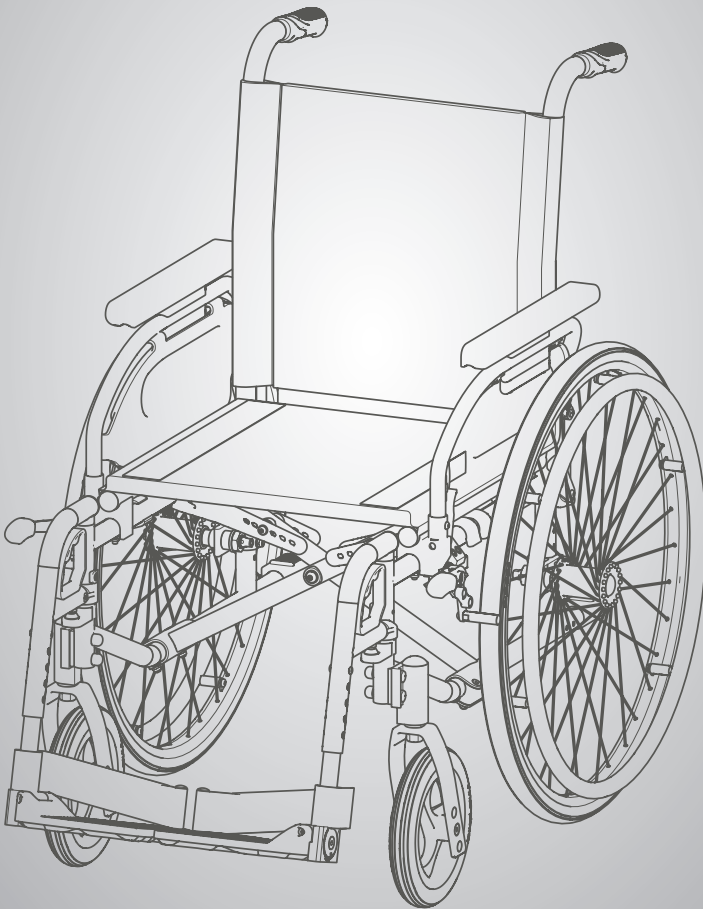
Bedienungsanleitung
Instruction manual
Manual de instrucciones

Pyro Start Plus Pyro Start Plus SL

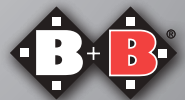
DE

EN

ES

Enjoy mobility.

Inhalt

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Notizen

This image shows a blank sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Notizen

[illegible]

Contents

1.	Introduction	50
1.1.	Labels and symbols	50
2.	Important safety information	51
2.1.	General safety information	51
2.2.	Risk of tipping over	53
2.3.	Safety in vehicles	53
2.4.	Use on roads	54
2.5.	Brakes	54
3.	Intended use	54
3.1.	Indications	55
3.2.	Contraindications	55
3.3.	Declaration of conformity	55
3.4.	Liability	56
3.5.	Service life	56
4.	Product and scope of delivery	57
4.1.	Checking the delivered goods	57
4.2.	Scope of delivery	57
4.3.	Product's identification label and serial number	58
4.4.	Overview	59
5.	Assembly/Adjustments	60
5.1.	Folding and unfolding	60
5.2.	Legrests	61
5.2.1.	Swing-away legrests	61
5.2.2.	Adjusting the legrests	62
5.3.	Angle-adjustable footplates	62
5.3.1.	Elevating legrests (optional)	62
5.4.	Amputee legrests (optional)	63
5.5.	Sideguards	63
5.5.1.	Standard sideguard	63
5.5.2.	Height and depth-adjustable sideguards (optional)	64
5.5.3.	Removable side panel (optional)	65
5.6.	Backrest height	65
5.7.	Seat angle	65
5.8.	Brakes	65

5.8.1.	Operating the parking brake	66
5.8.2.	Adjusting the parking brakes	67
5.8.3.	Operating the parking brake with the brake lever extension	68
5.9.	One-hand brake (optional)	68
5.10.	Drum brake (optional)	69
5.11.	Wheels and tyres	70
5.11.1.	Quick-release axle	70
5.11.2.	Drive wheels	70
5.11.3.	Castors	71
5.12.	Wheelbase extension	71
5.13.	Tipping aids	72
5.14.	Anti-tip wheels (optional)	72
5.15.	Headrest (optional)	73
5.16.	Adjustable backrest	73
5.17.	One-hand operation (optional)	73
5.18.	Reflectors	73
6.	Accessories	74
6.1.	Lap belt (optional)	74
6.2.	Therapy table (optional)	74
6.3.	Togo (optional)	74
6.4.	Crutch holder (optional)	74
6.5.	Spokeguard (optional)	74
6.6.	Handrim cover (optional)	75
6.7.	Infusion & oxygen bottle holder (optional)	75
6.8.	Fixing a bag (optional)	75
7.	Use	76
7.1.	Side transfer	76
7.2.	Frontal transfer	77
7.3.	Going up or down stairs or a high obstacle	78
7.4.	Driving the wheelchair	79

Pyro Start Plus | Lightweight wheelchair
Pyro Start Plus SL | Special sizes

8.	Technical data	80
9.	Transport	82
9.1.	Transporting the wheelchair	82
9.2.	Transportation in vehicles	82
10.	Maintenance instructions for the user	83
10.1.	Cleaning and disinfection	83
10.2.	Disinfection	83
10.3.	Checks before driving	84
10.3.1.	Checking the brakes	84
10.3.2.	Checking the tyres	85
11.	Repairs	85
12.	Maintenance information	85
13.	Maintenance instructions for the specialist trade	86
14.	Passing the wheelchair on to another user	86
15.	Storage/shipping	86
16.	Disposal	86
17.	Warranty	87

Lightweight wheelchair | **Pyro Start Plus**
Special sizes | **Pyro Start Plus SL**

1. Introduction

Dear user,

You have chosen a high-quality wheelchair from Bischoff & Bischoff. Thank you for trusting in us.

This user manual contains all the information necessary to operate the Pyro Start Plus / Pyro Start Plus SL wheelchairs.

The Pyro Start Plus SL model allows greater adaptability for larger individuals due to the larger seat height, seat depth and backrest height.

Your wheelchair was designed for both indoor and outdoor use.

Before using your new wheelchair for the first time, please read and follow the information in this user manual carefully. This is an integral and necessary part of the wheelchair. Keep the user manual within easy reach and pass it on when transferring the wheelchair.

For users who are visually impaired, this document is available as a pdf file on our website www.bischoff-bischoff.com.

Note!

Despite thorough research and editing, incorrect information may occur in our user manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the user manuals in the download area of our website:

www.bischoff-bischoff.com

1.1. Labels and symbols

Important! Indicates particularly useful information in the relevant context.

Attention! Means particular safety-related Notes. Observe the user manual!

2. Important safety information

2.1. General safety information

- To avoid falls and dangerous situations, practise using your new wheelchair on level, manageable terrain first. An accompanying person is recommended.
- Before using the wheelchair, check that all attached parts are properly fastened.
- Only lift the wheelchair by firmly mounted parts (see chapter 7.3).
- When shifting balance due to body movements or loading the wheelchair, the risk of tipping to all sides increases.
- Do not use the wheelchair to transport more than one person or loads. Observe the intended use (see chapter 3)!
- When adjusting and setting the wheelchair, there is a risk of trapping body parts!
- Keep clothing and body parts away from gaps between moving parts!
- When negotiating obstacles (steps, etc.), have an attendant present. Use the tipping aid (see chapter 5.14) (Fig. A).
- Only climb stairs and larger obstacles with the help of at least two escorts (see chapter 7.3). Use facilities such as ramps, lifts or stair-climbing aids - if available (Fig. B).
- When driving on a slope/gradient, note that there is a risk of tipping over from just a few degrees. The use of an anti-tipper (see chapter 5.14) is recommended (Fig. C).
- When getting in or out of the wheelchair, apply both parking brakes (see chapter 5.8.1).
- The brake function must be checked for proper operation before each journey.
- In road traffic, observe the road traffic regulations.

- Each time, before setting off, check the reflectors of your wheelchair to make sure they are not damaged and are visible for other road users.
- Only use the parking brakes when parking.
- The seating materials are frame resistant in accordance with EN 1021-2. However, please keep cigarettes and other possible ignition sources away from your product.
- Protect the wheelchair from sunlight to avoid burns caused by heated components
- Observe the storage and operating conditions (see Chapter 15).
- When transferring between the wheelchair and the bed (see Chapter 7.1 and 7.2) always
 - apply both parking brakes
 - flip footplate up or swing the legrests away.
- If the seat width/seat depth combination is inappropriate, this may lead to limitations in accessing escape routes.
- The dimensions of the Pyro Start Plus SL wheelchair exceed the recommended values in DIN EN 12183:2014/A.1 and PRM TSI Appendix M for dimensions and manoeuvring range.

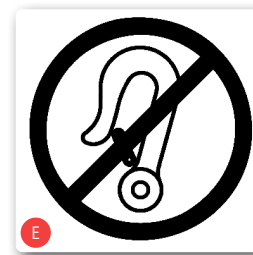
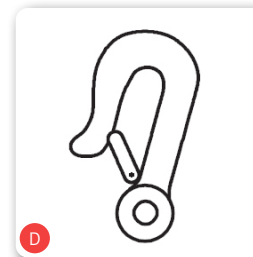
Attention: Due to the dimensions of the wheelchair, it is necessary to check whether your wheelchair can also be moved in the planned environment. This applies in particular to the use of escape routes.

- Reporting of incidents.
 If you, as the operator, user or their relatives, discover possible defects or functional limitations, please contact an authorised specialist dealer immediately. Operators, users or their relatives should inform their specialist dealer who provided them with the product of any suspected serious incidents that have directly or indirectly had one of the following consequences or could have and by which they are affected. They can also report this to the competent higher federal authority.
- The death of a patient, user or other person,
- The temporary or permanent severe,

- Deterioration of the health condition of a patient, user or other person,
- A serious threat to public health.

Protect the wheelchair from sunlight to avoid burns caused by heated components

2.2. Risk of tipping over

If fitted, the anti-tip wheels which are fitted on both sides (see Chapter 5.15) will largely prevent the wheelchair from tipping over backwards. However please note that driving with anti-tip wheels is only possible in a limited way on ground with different levels, lift platforms and ramps. Make sure there is sufficient gap at the top, bottom and sides.

2.3. Safety in vehicles

To see whether your wheelchair is permitted for use as a seat in a vehicle for transporting individuals with impaired mobility or not, you can refer to the symbols on the product's identification label (see Chapter 4.3).

Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. D).

Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. E).

If your wheelchair is not designed to be used as a seat in a vehicle, under no circumstances must it be used as such. It does not fulfil the requirements in compliance with ISO 7176-19. Violation can cause the most serious injuries resulting in death in the case of an accident (Fig. E).

Pyro Start Plus	May be used as a vehicle seat
Pyro Start Plus SL	Must not be used as a vehicle seat

2.4. Use on roads

The wheelchair is designed for indoor and outdoor use. Please note that when you drive on public roads you must comply with road traffic regulations. Do not endanger other users through reckless driving, this applies particularly on pavements.

2.5. Brakes

Operating brake:

When driving, brake the wheelchair using the handrims. Please note that your hands may heat up during this process. (see chapter 5.8).

Parking brake/parking:

As soon as the wheelchair is stationary, push both brake levers forward.

The wheelchair is safely braked when it can no longer be pushed away and both brake levers are pushed forward as far as they will go. Pull the lever back towards you to release the brake (see chap. 5.8.1).

Drum brake:

The wheelchair can optionally be equipped with a drum brake, that can only be operated by an attendant via the brake levers on the push handles.

3. Intended use

The wheelchair is designed to increase the mobility only for grown up people who have difficulty walking with the indications specified (see Chapter 2.6). The maximum user weight for the:

Pyro Start Plus of 125 kg

Pyro Start Plus SL of 150 kg

must not be exceeded.

Please note:

We can only provide a warranty if the product is used under the conditions specified and for the intended purposes.

3.1. Indications

Provision of a wheelchair is intended for grown up people who are unable to walk or who have severely impaired mobility, e.g. through:

- paralyses
- loss of limbs
- limb defects/deformity
- joint contractions (not on both arms)
- joint damage (not on both arms)
- other illnesses

Provision of an SL wheelchair may be considered if the height of the user is greater than 185 cm and a standard wheelchair can no longer provide the adjustability required.

3.2. Contraindications

Provision of wheelchairs is not suitable for individuals who have:

- no capability to sit
- joint damage/contractions on both arms
- perceptual disorders
- severe equilibrium disorders
- loss of limbs on both arms
- reduced and insufficient vision
- severe limitations of cognitive abilities

3.3. Declaration of conformity

We, as Bischoff & Bischoff GmbH, declare under sole responsibility, that the Pyro Start Plus /Pyro Start Plus SL wheelchair meets all requirements of MDR 2017/745, which are applicable.

3.4. Liability

We can only provide a warranty if

- the product is used under the conditions specified and for the intended purpose.
- Modifications, enhancements, repairs and maintenance work is carried out only by those individuals who have been authorised by us to do so, and
- the wheelchair is used in accordance with the instructions for use.

3.5. Service life

The expected product service life is up to five years. Prerequisites for this are proper use as well as compliance with the safety, care and maintenance information in accordance with this user manual.

4. Product and scope of delivery

4.1. Checking the delivered goods

All products from Bischoff und Bischoff GmbH undergo an appropriate final inspection at our factory and the CE mark is affixed (see Chapter 3.3).

The wheelchair will be delivered in a special cardboard box. After unpacking, retain the cardboard box if possible. It can be used for possible later storage or return of the product.

Delivery and induction is usually carried out by a qualified medical equipment retailer.

When sending by train or shipping company, the goods must be checked immediately for transport damage in the presence of the forwarding agent.

Check that the delivery is complete and undamaged (see Chapter 3.2).

In the case of anomalies or damage, contact our Customer service. You can find the contact details on the back cover of this user manual.

4.2. Scope of delivery

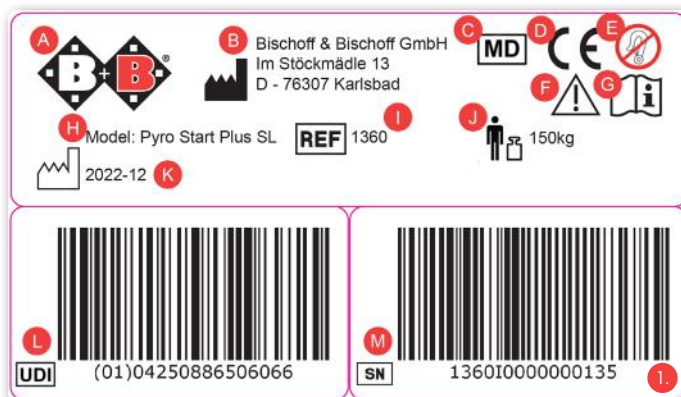
On receiving the goods, please immediately check that the delivery is complete. The delivery consists of:

- packing,
- wheelchair (preassembled),
- legrests
- this user manual
- any necessary accessories

4.3. Product's identification label and serial number

The product's identification label (Fig. 1) and serial number are located on the cross brace.

- A** Manufacturer's logo
- B** Manufacturer information
- C** MD Medical product
- D** CE mark
- E** Observe safety in vehicles
- F** Attention! - read the User manual
- G** Important! - read the User manual
- H** Model name
- I** Model number
- J** Max. payload
- K** Date of manufacture
- L** UDI number
- M** Serial number


Depending on the model number, the information may vary. Refer to your product's identification label for the information which relates to your product.

4.4. Overview

(see Fig. 2)

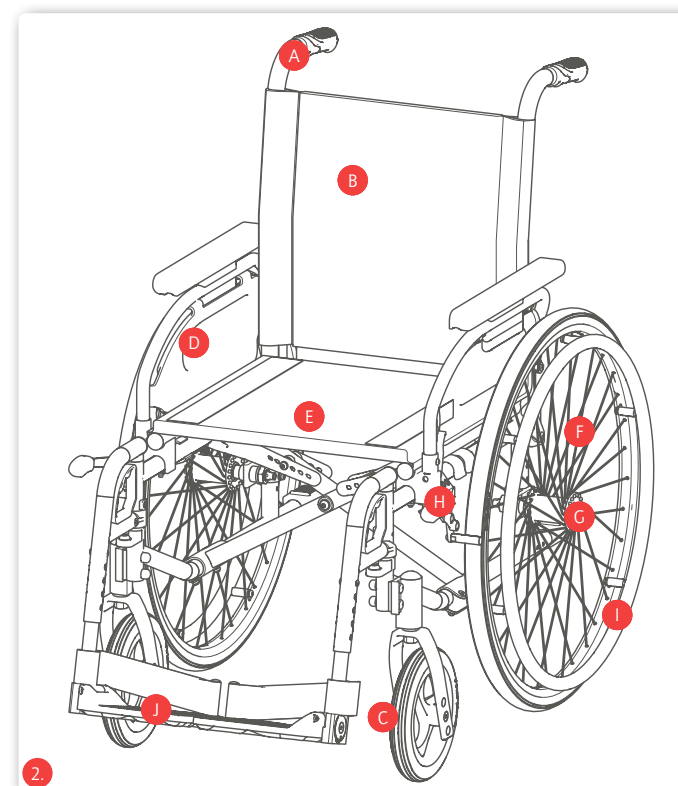- A** Push handle
- B** Adjustable backrest and upholstery
- C** Castor
- D** Sideguard
- E** Seat sling
- F** Drive wheel
- G** Quick-release axle
- H** Parking brake
- I** Handrims
- J** Legrest with heel strap


5. Assembly/Adjustments

Your new Bischoff & Bischoff wheelchair will be delivered in a Bischoff & Bischoff original cardboard box, assembled and folded.

In order to avoid transport damage, detachable attachments are delivered separately with the product.

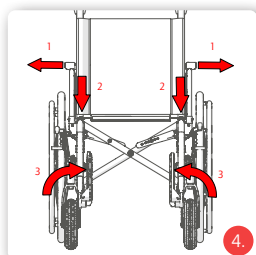Refer to the technical data to see which options are possible on your wheelchair (see Chapter 7).

5.1. Folding and unfolding

Unfolding

- Position the folding wheelchair next to you. (Fig. 3)
- Tip the wheelchair towards you until the weight is off one of the drive wheels.
- Push the seat tubes apart and push them down with the flat of your hand, until they lock into place in the bracket on the frame (Fig. 4).
- The seat pad must be completely unfolded.
- Apply the left and right parking brakes on the drive wheels.
- Fit the legrests.
- You can now sit down

Folding

- Position the folding wheelchair next to you.
- Flip the footplates up (Fig. 5)
- Remove the seat cushion if one is fitted, take hold of the front and rear of the seat upholstery and pull it up.

When folding and unfolding, watch your fingers as there is a risk of pinching.

5.2. Legrests

5.2.1. Swing-away legrests

You can flip the footplates up separately to the side and adjust to the lower leg length of the user. You can remove the legrests, swing them away and refit them without needing to use tools.

Attach legrests

- Hold the top of the legrest so that it is pointing outwards.
- Guide the legrest into the hole in the frame using the guide bolts (Fig. 6).
- Swing the legrest to the front until you hear it clearly click into place (Fig. 6)
- Flip the footplate down if necessary.

Swing the legrests away and remove them

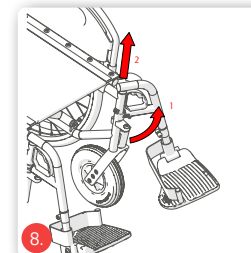
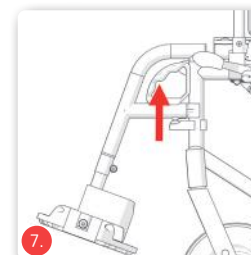
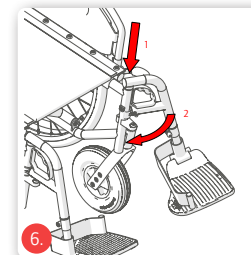
In order to make it easier to transfer to and from your folding wheelchair, you can swing the legrests inwards or outwards, without having to remove them.

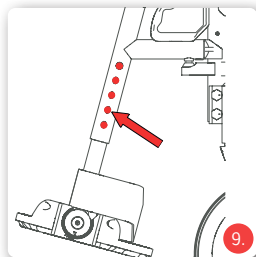
- Flip the footplate up if necessary.
- Pull the handle upwards (Fig. 7) and swing the legrest outwards (Fig. 8).
- In the outward position, the legrest can be pulled out by pulling it upwards (Fig. 8).

Before setting off, fit the legrests as described above.


Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching.

5.2.2. Adjusting the legrests

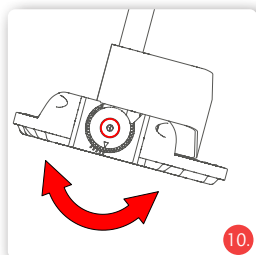
Loosen the screws, then adjust the legrest to the required length and re-tighten the screws (Fig. 9).

5.3. Angle-adjustable footplates

The angle of the footplates can be adjusted:

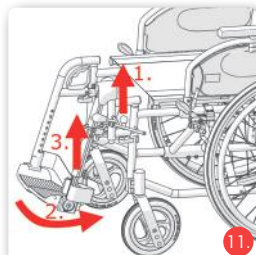- Unscrew the hexagon screw at the sprocket (see Fig. 10) Move the footplate to the required position by rotating the sprocket.
- Re-tighten the sprocket screw (see Fig. 10).

5.3.1. Elevating legrests (optional)

Assemble, disassemble and swing them away as for the standard legrests.

To adjust, undo the clamp lever, then adjust the legrest to the required length and then tighten the clamp lever.

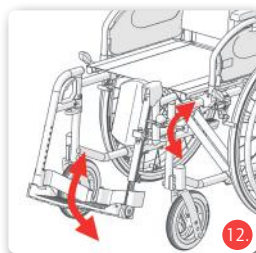
The horizontal adjustment works as follows:

Upwards

- by lifting the footplate receiver tube you can get the legrest into the required position.

Downwards

- Pull locking lever upwards
- When doing this, hold the legrest firmly and move to the required position
- Release locking lever; legrest locks into place automatically.

Never stand on the footplate and when swinging it away, watch your fingers as there is a risk of pinching

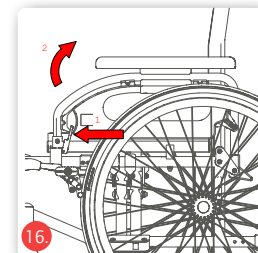
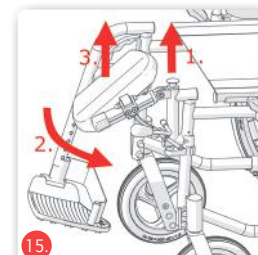
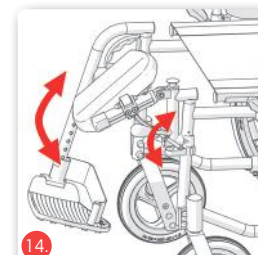
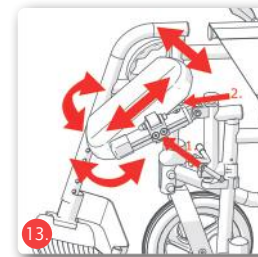
To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

5.4. Amputee legrests (optional)

Assembly is the same as for the standard leg supports (chapter 5.2.1.). The lower leg length can be adjusted once by loosening the screws on the side of the amputation leg support (1.) and moving the calf pad to the desired position. There is another possibility to adjust the lower leg and thigh length by remounting the calf pad with the socket screws on the back. For horizontal adjustment, simply pull the calf pad upwards. To reset, pull the clamping lever (Fig. 14). Once the correct position has been reached, release the clamping lever. The calf pad can also be swivelled without tools and folded outwards to save space (Fig. 13). Disassembly (Fig. 15) is done by pulling the knob (1.), then swinging it out (2.) and pulling the amputation leg support upwards (3.).


To change seat, the legrest must be swung away or removed. Never sit on the legrest. Risk of tipping over!

5.5. Sideguards

5.5.1. Standard sideguard

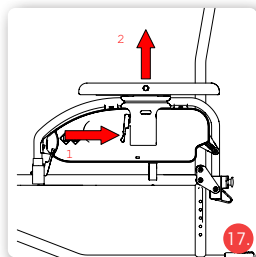- In order to be able to transfer more easily, both sideguards with armrests can be swung backwards.
- Pull the locking lever gently and at the same time, swing the sideguard to the back (Fig. 16).
- To lock the sideguard, flip it forwards and click it into place on the armrest using gentle pressure.
- The arm pads on the sideguards are fitted in the central position when delivered.
- They can be fitted in either the front or rear positions.
- The sideguard can be height-adjusted in four steps, to do this remove the screw under the armrest, mount the spacers in such order reaching the desired height and re-tighten.

When swinging the sideguards away, make sure that clothing or other items cannot be trapped. Before using for the first time, check that the sideguards are firmly in place!

Never use the sideguards to carry the wheelchair!

5.5.2. Height and depth-adjustable sideguards (optional)

As an option, your wheelchair can be fitted with height and depth-adjustable sideguards. The sideguard can be removed and swung away.

Height adjustment

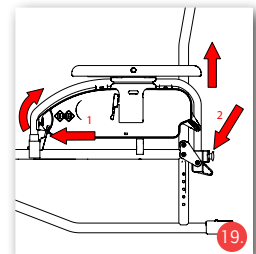
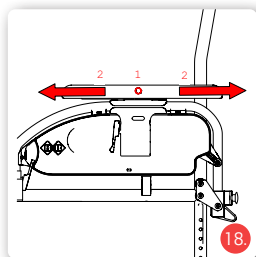
- Pull the locking lever to move the armrest to the required position (Fig. 17).
- When you have reached the position required, then release the lever. The lock clicks into place.
- The height has been correctly selected if the upper arm and forearm are approximately at right angles to one another when the forearm is resting on the armrest.

Depth adjustment

- Press the knob on the side of the armrest and push the armrest to the required position (Fig. 18).
- Release the knob and move the armrest gently forwards or backwards until it clicks into place.


Remove side panel

To remove the side panel, first swing the side panel upwards by pressing button 1. The side panel can then be pulled upwards by pulling button 2 (Fig. 19).

5.5.3. Removable side panel (optional)

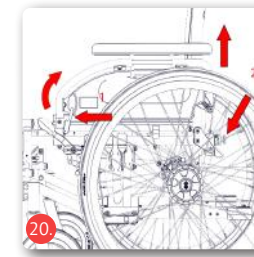To remove the side panel, first remove the side panel by pressing the button 1 upwards. The side panel can then be pulled upwards by pulling button 2 (Fig. 20).

5.6. Backrest height

The Pyro Start Plus SL backrest is height adjustable.

Adjusting the backrest height must only be carried out by trained personnel.

5.7. Seat angle

The seat angle can be adjusted by adjusting the height of the front and rear wheels. See chap. 5.11.1 and 5.11.2

5.8.Brakes

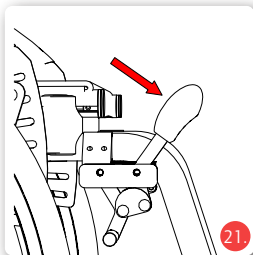
As standard, the wheelchair is fitted with parking brakes to be operated by the wheelchair user.

The parking brakes must not be used as operational brakes, as there is a risk of death due to the risk of falling!

Please note that there is a risk of burns if your hands heat up when braking using the handrims!

5.8.1. Operating the parking brake

The parking brake must only be used for parking. To slow down while driving, you must only use the drive wheel handrims or the optional drum brakes for attendant.

- When braking using the handrims, hold the highest point. First let the handrims slide through your hands.
- Slowly increase the braking force by exerting gripping force on the handrims, until you come to an abrupt stop.
- As soon as the wheelchair has stopped, push both brake levers forward (Fig. 21). The wheelchair is securely braked when it can no longer be pushed and both brake levers are pushed forward to the stop position.

Pull the lever towards you to release the brake.

Never use the brake lever as a support - e.g. when transferring or standing up -, otherwise there is a risk of tipping over and falling! The lever could break off!

The parking brake - with or without brake lever extension - only works on one drive wheel and must therefore always be used on both sides!

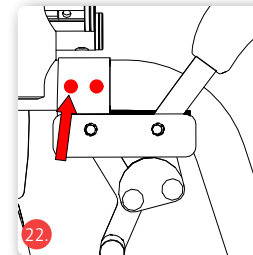
5.8.2. Adjusting the parking brakes


Adjusting the brakes must only be carried out by trained personnel. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

The drum brake does not replace the parking brake.

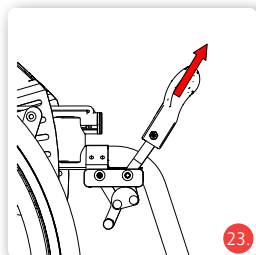- Mount the brake in a position which is operable and offers secure braking in all driving situations.
- To adjust the gap, undo the two socket screws on the brake bracket (Fig. 22).
- Then push the brake system to the correct position along the bracket.
- Finally re-tighten the two socket screws.
- Make sure that the brakes click into place when operating the lever and that they don't snap back.


5.8.3. Operating the parking brake with the brake lever extension

The procedure for operating the parking brake with the brake lever extension is the same as operating the parking brake (see Chapter 5.8.1).

Folding the brake lever extension in

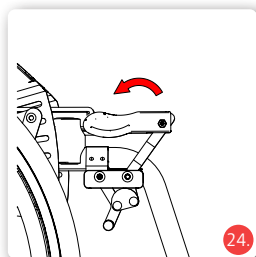- The brake lever can be folded in when driving or transferring to the side.
- Pull the plastic grip out of the lever (Fig. 23).
- Fold the plastic grip down (Fig. 24).
- Before setting off, the brake must be returned to its operational position by following the procedure in reverse order.

Never use the brake lever as a support - e.g. when transferring or standing up -, otherwise there is a risk of tipping over and falling! The lever could break off!

The parking brake - with or without brake lever extension - only works on one drive wheel and must therefore always be used on both sides!

5.9. One-hand brake (optional)

Using the one-hand brake, you have a parking brake which you can use to brake both drive wheels using a brake lever.

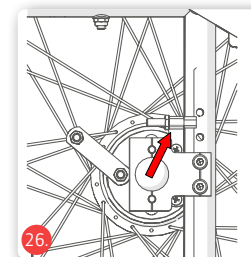
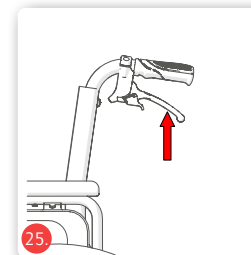
5.10. Drum brake (optional)

As an option, the wheelchair can be fitted with a drum brake which can only be operated by an attendant using the brake lever on the push handles (Fig. 25).


Adjusting the drum brake

Basic setup for the drum brake is carried out on the cable clamping screw. Fine adjustment is carried out using the adjusting nut on the brake cable. Unscrewing the adjustment nut will increase the braking effect (Fig. 26).

- Drum brakes only work on one drive wheel at a time!
- Drum brakes are also subject to natural wear. If the braking effect decreases, the wheelchair must no longer be used. In order to avoid this happening, for your own safety you should check that the brakes are working properly each time before setting off!
- Once you have finished the adjustment, re-tighten all screws and carry out a brake test!

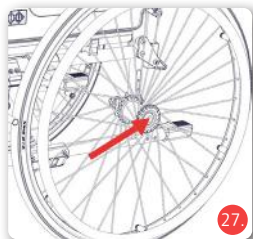Adjusting the brakes must only be carried out by trained personnel. Please bear in mind that if the brakes are set incorrectly this can have life-threatening consequences.

5.11. Wheels and tyres

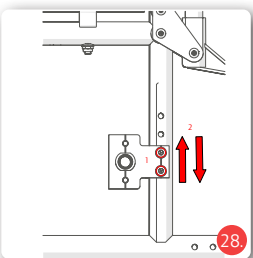
The seat height can be adjusted to 3 positions to meet the user's requirements.

5.11.1. Quick-release axle

As standard the wheelchair has quick-release axles. This means you can easily remove and refit the drive wheels.

- Push the lock of the quick-release axle using your thumbs and pull the wheel off (Fig. 27).
- When refitting, simply put the wheel onto the receiver. When doing this, push the locking knob in.

The click you will hear from the quick-release axle will indicate that the wheel is securely mounted. After fitting, make sure that the wheels are tight!

5.11.2. Drive wheels

- Remove the drive wheels using the quick-release axles.
- Undo the screws on the adapter and pull them off completely (Fig. 28).
- Position the adapter to the required seat height.
- Insert the screws back into the adapter and re-tighten the screws.

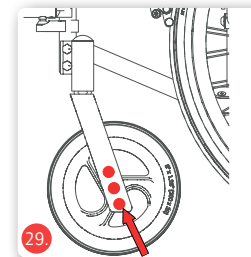
If your wheelchair has the optional pneumatic tyres, pump the wheel to the pressure according to the value on the tyre's wall.

Please contact your dealer in case of puncture in your pneumatic tyres.

5.11.3. Castors


- Undo the axle and remove it completely.
- Position the castor in the position for the required seat height (Fig. 29).
- Put the axle back into the castor fork and tighten.
- Alternatively, the height can also be adjusted by rotating the castor bearing bracket. Please contact your dealer to do this.
- The screw which is used to adjust the play in the steering head is located under the cap at the upper end of the castor fork.


If the drive wheel or castor is worn or defective, ask your dealer to service them!

5.12. Wheelbase extension

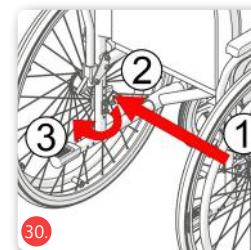The wheelchair's stability and steadiness is improved by extending the wheelbase. Conversely, if the wheelbase is reduced, the manoeuvrability is improved. Reducing the wheelbase is only recommended for experienced riders. To adjust the wheelbase, undo the adapter which is held with two screws and then move this backwards and re-tighten (pic. 30). Make sure that all screws are firmly tightened.

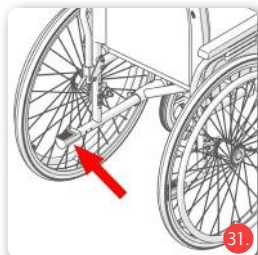
Attention: risk of tipping over! If you move the drive wheel forwards, then the risk of tipping over increases significantly. Anti-tip wheels or a wheelbase extension are required. Check the settings first with an assistant. Only use settings which you consider to be safe.

If the wheelbase is changed, then it is essential to adjust the parking brake as well!

5.13. Tipping aids

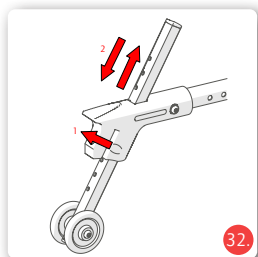In order to make it easier for an attendant to negotiate obstacles such as kerb edges, tipping aids can be used. Hold the wheelchair firmly with both hands on the push handles. Press one foot onto one of the tipping aids and at the same time push the hands down (Fig. 31). Tipping may be restricted by the anti-tip wheels.

5.14. Anti-tip wheels (optional)

The anti-tip wheels prevent the wheelchair from tipping over backwards. There must be a gap between the rollers of the anti-tip wheels and the ground of approx. 30-50 mm (Fig. 32).

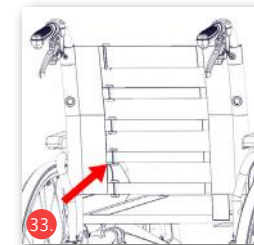
Anti-tip wheels are particularly recommended for inexperienced wheelchair users or amputees.

For safety reasons, always use the anti-tip wheels as a pair!

5.15. Headrest (optional)

The headrest is mounted on the push handles. After undoing the clamp lever or the star screw you can adjust the headrest to the right position. Then re-tighten the screws.

5.16. Adjustable backrest

To enable you to sit comfortably, the hardness of the backrest can be adjusted.

The tighter you adjust the backrest, the harder the backrest feels. First remove the backrest upholstery that is attached with a Velcro fastener. Then you can tighten or loosen the individual straps (Fig. 33).

To relieve the strain on certain areas of your back, tighten the straps a little looser. Now you must fasten the backrest upholstery again with the Velcro fastener.

5.17. One-hand operation (optional)

With one-hand operation you have both handrims on one wheel, so that you can operate both drive wheels with one hand. The option can be fitted either left or right.

5.18. Reflectors

As standard your wheelchair is fitted with reflectors on the back and spokes. These must not be covered.

6. Accessories

6.1. Lap belt (optional)

You can order a lap belt as an option and this will secure users who do not have the required stability in the wheelchair.

- Remove the sideguard or swing it away (see Chapter 5.5).
- On both open ends, wrap the belt once round the sideframe tube.
- Then fix the belt strap with the buckle.
- You can adjust the belt length using the sliding tabs on the belt.
- Use the fastening clip to open and close the belt.

6.2. Therapy table (optional)

The table is simply pushed into the armrest from the front. To remove the therapy table, simply pull it back out.

6.3. Togo (optional)

An electrical Togo pushing & braking aid can be fitted to your wheelchair.

If required, see the Togo pushing & braking aid user manual for all relevant information on assembling and use.

6.4. Crutch holder (optional)

Put the crutch in the crutch holder and then fix it to the backrest using the retaining strap.

Never step on the crutch holder and never use it as a tipping aid.

6.5. Spokeguard (optional)

The spokeguard is fixed to the spokes using the cable ties.

6.6. Handrim cover (optional)

Put the handrim cover over the handrims in order to achieve better grip on the handrim, for example if your hand strength is limited.

With better grip, there is a risk of sharp braking.
Attention: risk of falling!

6.7. Infusion & oxygen bottle holder (optional)

The infusion & oxygen bottle holder is pushed onto the back tube and fixed to the backrest of your wheelchair.

Attention - with increased load from the oxygen bottles, the risk of your wheelchair tipping over backwards will also increase. We strongly recommend the use of anti-tip wheels.

6.8. Fixing a bag (optional)

If you want to fit a bag to your wheelchair, hang the straps of your bag over the back tubes of your wheelchair.

Please note that the reflectors must not be covered.

Attention - with increased load in the bag, the risk of your wheelchair tipping over backwards will also increase. We strongly recommend the use of anti-tip wheels.

7. Use

7.1. Side transfer

Never stand on the footplate and when swinging it away, there is a Risk of tipping over!

- Make sure that the sides of the wheelchair and the current or future seat surface are as close to each other as possible.
- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Secure the wheelchair using the parking brake (see Chapter 5.8).
- If necessary, swing the armrest backwards or remove the sideguard (see Chapter 5.5).
- Flip both footplates up, swing them to the side and remove them (see Chapter 5.2).
- Slide sideways onto the other seat surface. When doing this sit as far back as possible.
- Move the legrests back into their original position.
- If necessary lock the sideguards back in place.


7.2. Frontal transfer

Never stand on the footplate and when swinging it away, there is a Risk of tipping over!

- Remove the legrests (see Chapter 5.2).
- Make sure that the front of the wheelchair and the current or future seat are as close to each other as possible and apply the parking brakes on both sides (see Chapter 5.8).

Pyro Start Plus | Lightweight wheelchair Pyro Start Plus SL | Special sizes

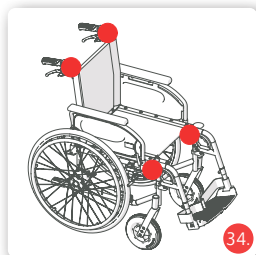- If you are transferring from another wheelchair or indoor frame, secure these by applying the parking brakes.
- Now slide onto the seat surface by rotating your body.
- Then hang the legrests back into the initial position and flip the footplates down.

7.3. Going up or down stairs or a high obstacle

Only the frame and the push handles must be used as grip points.

The helpers only grip on the firmly fixed frame parts (Fig. 34). In order to go up stairs, the rear helper pulls the wheelchair backwards up the steps by holding onto firmly-mounted handles. The lower helper grips on the front frame tubes and stabilises the wheelchair. When doing this he pushes the drive wheels of the wheelchair onto the steps.

When descending, the lower helper brakes the chair by pressing the wheelchair straight into the steps. The upper helper holds the wheelchair by the push handles, keeps it secure and holds it in the right position. The wheelchair should roll down step by step.

Two attendants must help with obstacles and stairs with more than three steps.

Lightweight wheelchair | Pyro Start Plus Special sizes | Pyro Start Plus SL

7.4. Driving the wheelchair

- Sit in your Bischoff & Bischoff wheelchair. Make sure that you sit as far back as possible.
- Propulsion is achieved using the handrims on the drive wheels. Place your hands on the handrims using your thumbs and bent index fingers.
- From this position, push the wheelchair with both hands (Fig. 35).
- Turning right: hold the right handrim tight and only push with the left one
- Turning left: push in the opposite way
- Turning on the spot: push the handrims in opposite directions at the same time
- Braking: reduce the speed of the handrims with your hands.

Carry out the first test drives carefully, until you have got used to the wheelchair and its driving characteristics. Wheelchairs only have limited security against tipping over and slipping. Take extra care when braking, starting off or turning on slopes. Extended leaning out of the wheelchair also increases the risk of tipping over. Tipping protection reduces this risk. For easier obstacle negotiation, the attendant can tip the wheelchair (see Chapter 5.13).

8. Technical data

	Pyro Start Plus	Pyro Start Plus SL
Dimensions and weights		
Total length with legrests (mm):	1050	1100
Total length without legrests (mm):	800	860
Total width (mm):	SW + 200	
Total width folded (mm):	295	
Total width folded with drum brake (mm):	320	
Total height (mm):	920	985
Push handle height (mm):	920	985
Lower leg length (mm):	420, 460, 480, 500	460, 480, 500, 520, 540, 560, 580, 600
Kerb weight (kg):	14.2	15.8
Max. payload (kg):	125	150
Seat width (mm):	370, 400, 430, 460, 490, 520	430, 460, 490, 520
Seat depth (mm):	400, 420, 440	460, 480, 500, 520
Front seat height without seat cushion (mm):	450, 475, 500	525, 550, 575, 600
Rear seat height without seat cushion (mm):	415, 435, 455, 475	500, 520, 540, 560, 580
Seat angle°:	0 - 4,5	
Default backrest angle at initial delivery (°):	98	
Backrest height without seat cushion (mm):	420, 440	480, 500, 520
Armrest height without seat cushion (standard side-guard) (mm):	190, 205, 220, 235	235, 250, 265, 280

	Pyro Start Plus	Pyro Start Plus SL
Armrest length (mm):	340	
Armrest width (mm):	40	
Turning area (mm):	1040	1140
Static stability (°):	7	
Parking brake operating force (N)	< 60	
Tyres		
Drive wheels (inch):	24" PU	
Transit wheels	12" PU	
Castors (mm):	8" x 1.25" PU	
Material		
Frame:	Aluminium	
Seat and backrest:	Nylon	
Storage conditions		
Temperature:	-40 - +65°C	
Humidity:	45 - 60%	
Operating conditions		
Temperature:	-25 - +50°C	
Humidity:	20 - 80%	

The dimensions stated are subject to manufacturing tolerances of +/- 10 mm.

9. Transport

9.1. Transporting the wheelchair

In order to make the wheelchair as manageable as possible to transport, remove all removable parts (legrests, drive wheels, armrests, loose accessories, see Chapter 5) and fold the wheelchair up (Fig. 36).

9.2. Transportation in vehicles

To see whether your wheelchair is permitted for use as a seat in a vehicle for transporting individuals with impaired mobility or not, you can refer to the symbols on the product's identification label (see Chapter 4.3).

Wheelchair is designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. 37).

Wheelchair is not designed for use as a seat in a vehicle for transporting individuals with impaired mobility (Fig. 38).

For any queries regarding how to secure your product in vehicles for transporting individuals with impaired mobility, please contact Bischoff & Bischoff Customer Service. You can find the contact details on the back cover of this user manual.

If your wheelchair is not crash-tested, it must not be used as a vehicle seat under any circumstances. There is a risk of death!

10. Maintenance instructions for the user

10.1. Cleaning and disinfection

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.

- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
Do not use a high-pressure cleaner.
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

10.2. Disinfection

Before disinfection, the rollator must be thoroughly cleaned. The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants.

- In the case of multiple product users, a commercial disinfectant is prescribed.
- Use only wipe-on disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Verbund

für Angewandte Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.

- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by disinfectants. Longer-term restrictions of the functionality can be the result.

Please note our enclosed overview of the most important care, hygiene and maintenance tasks.

Attention: Danger of infection! If there are several users, the wheelchair must be cleaned after each use.

10.3. Checks before driving

10.3.1. Checking the brakes

Before setting off each time, check that the brakes work properly.

With the parking brakes applied, the wheelchair cannot be pushed on dry surfaces. When the brakes are released, the wheelchair runs without scraping noises and good directional stability.

If this is not the case, notify your dealer immediately and stop using the product until it has been serviced.

10.3.2. Checking the tyres

Each time before setting off, check the tyres for damage. If you have pneumatic tyres adjust to the air pressure given on the tyres.

11. Repairs

Repairs and servicing work must only be carried out by authorised partners (medical supply stores or dealers). To find a partner in your area, please contact our Customer Service. You can find the contact details on the back cover of this user manual.

12. Maintenance information

Typical wear parts are brakes and tyres. Please contact your dealer for spare parts.

We recommend that you have your dealer check your wheelchair at least once a year.

If there are faults or defects, pass the wheelchair to your dealer for servicing immediately.

On request, any information and documents necessary for the repair and servicing will be provided to the dealer. If you have any questions please contact Bischoff & Bischoff Customer Service. You can find the contact details on the back cover of this user manual.

The wheelchair must be checked in accordance with the maintenance plan and be in proper technical condition.

13. Maintenance instructions for the specialist trade

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance plan. A maintenance schedule will be provided upon request. A training programme for the specialised trade is also available.

14. Passing the wheelchair on to another user

The wheelchair is designed for re-use. Before it is passed on to another user, it must be serviced by a dealer and hygienically cleaned. When passing the wheelchair on to another user, please remember to hand over this user manual and the address of your dealer to the new user.

15. Storage/shipping

If the wheelchair is to be stored or shipped, all loose parts which are not fixed, except for the drive wheels, must be removed and packed individually in an appropriate cardboard box. The individually-packed parts can then be packed together in a larger cardboard box. We recommend that you keep the original packing and store it for these purposes, so that it is available when needed. Your wheelchair is then optimally protected against environmental influences during storage or transportation.

16. Disposal

If your wheelchair is no longer in use and needs to be disposed of, contact your dealer.

If you want to handle the disposal yourself, find out about the regulations in your area from local recycling companies.

17. Warranty

1. For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.

2. This warranty excludes claims for the following reasons:

a) normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.

b) overloading the product such as, for example, by exceeding the maximum user weight or load.

c) the product or part has not been maintained or serviced in accordance with the operating manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.

d) Accessories have been used which are not original accessories.

e) The product or part has been damaged through negligence, accident or incorrect use.

f) changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.

g) Repairs have been carried out, before our customer service has been informed of the circumstances.

3. To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.

4. For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).

5. For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your qualified mobility dealer will be happy to provide these to you.

Note!

Despite thorough research and editing, incorrect information may occur in our User manual. We reserve the right to make technical modifications. Images may show accessories. You can find the current version of the User manuals in the download area of our website:

www.bischoff-bischoff.com

Warranty certificate

Product: Pyro Start Plus /Pyro Start Plus SL

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and stamp

(Pay attention to our general terms and conditions.)

*Signs are located on the cross brace

Failure to observe the instructions in the User manual as well as not carrying out maintenance work properly as well as, in particular, making technical modifications and additions (attachments) without the agreement of Bischoff & Bischoff GmbH will lead to loss of warranty and the manufacturer's liability.

Pyro Start Plus | Lightweight wheelchair
Pyro Start Plus SL | Special sizes

Notes

[illegible]

Lightweight wheelchair | **Pyro Start Plus**
Special sizes | **Pyro Start Plus SL**

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